# Device and Update Manager

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PC software for ACCUVAC Pro

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Software de PC para ACCUVAC Pro

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# Contents

<b>1</b>	<b>Introduction</b>	<b>15</b>
1.1	Product description .....	15
1.2	System requirements .....	15
<b>2</b>	<b>Safety</b>	<b>15</b>
<b>3</b>	<b>Description</b>	<b>16</b>
<b>4</b>	<b>Preparation and operation</b>	<b>17</b>
4.1	Install Device and Update Manager .....	17
4.2	Connect the device to the PC .....	17
4.3	Start Device and Update Manager .....	19
4.4	Read out device data .....	19
4.5	Perform firmware update .....	20
4.6	Read out service data .....	21
4.7	After use .....	21
<b>5</b>	<b>Faults</b>	<b>23</b>

# 1 Introduction

## 1.1 Product description

You can use the Device and Update Manager PC software to read out device and service data from ACCUVAC Pro and to perform firmware updates on ACCUVAC Pro.

If you have any questions or feedback, please contact our Technical Service team on:

telephone: +49-(0) 40/88 18 96-122

e-mail: [AfterSalesService@weinmann-emt.de](mailto:AfterSalesService@weinmann-emt.de)

## 1.2 System requirements

To use Device and Update Manager, you will need a PC with the following minimum configuration:

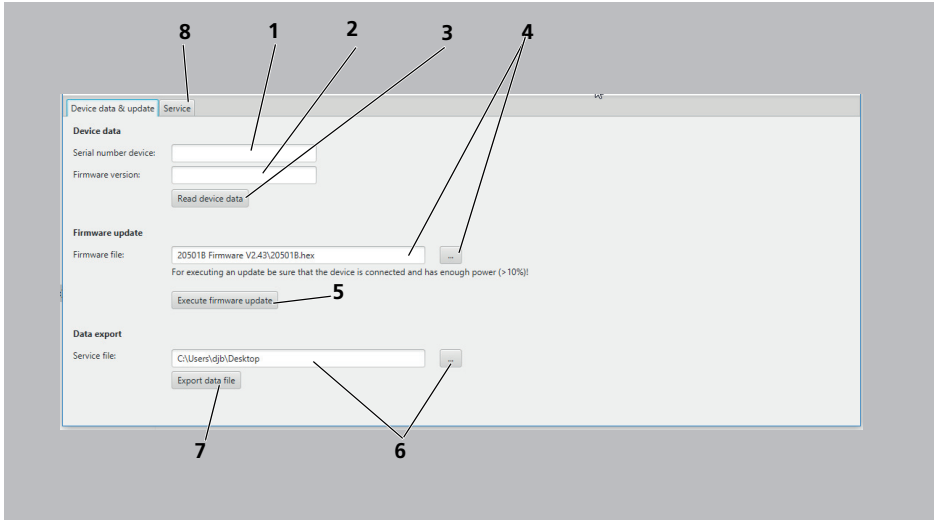
- Operating system: Microsoft® Windows® 10, Microsoft® Windows® 7  
(Microsoft, Microsoft Windows, Windows, Windows 10 and Windows 7 are registered trademarks of the Microsoft Corporation in the USA and/or other countries)
- A current version of JAVA
- USB port (type A connector)
- USB cable (type A connector to type mini-B connector)

# 2 Safety

Read these Instructions for Use carefully. Also observe the instructions for use for ACCUVAC Pro.

# 3 Description

## Overview of control panel

3-1 Overview of control panel (example)

No.	Designation	Description
1	Serial number device	Indicates the serial number.
2	Firmware version	Indicates the current firmware version of the device.
3	Read device data	For reading out device data.
4	Firmware file	Storage location (example) for the firmware update and button for selecting storage location
5	Execute firmware update	For updating the device firmware.
6	Service file	Storage location (example) for the service file with service data and button for selecting storage location
7	Export data file	For reading out service data.
8	Service	Area for specialist staff authorized by WEINMANN Emergency

## 4 Preparation and operation

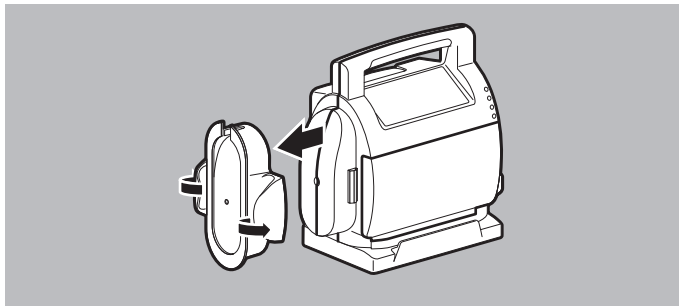
### 4.1 Install Device and Update Manager

- Requirements*
- Windows® is installed on the PC (see "1.2 System requirements", page 15).
  - A current version of JAVA is installed on the PC.
1. Download Device and Update Manager from the Download area of the WEINMANN Emergency home page to any storage location.
  2. Unzip the downloaded zip file.  
The tool will be in the selected storage location in the form of a file called **Device and update manager.exe**.


*Result* Device and Update Manager is installed.

### 4.2 Connect the device to the PC


- Requirements*
- The device has a battery capacity > 10 %.
  - The device is switched off.

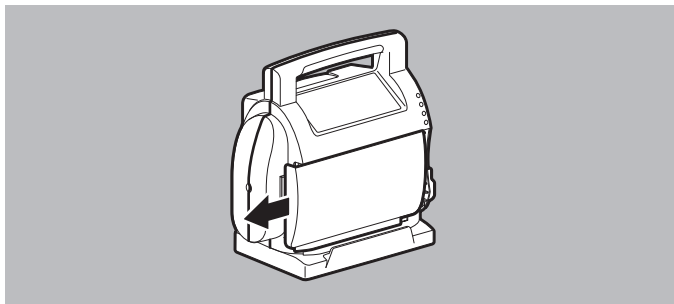
1. Using your thumbs, push apart both wings of the holder for suction tube on the device and take it off the device.

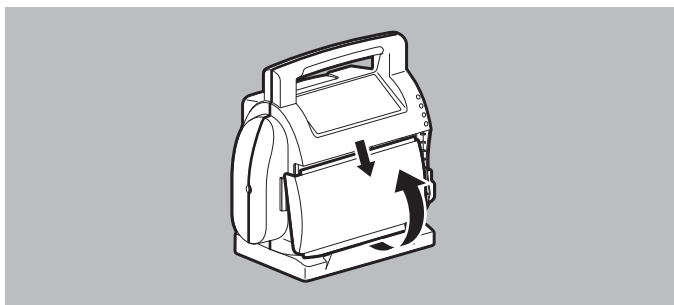2. Push the battery compartment cover to the left.

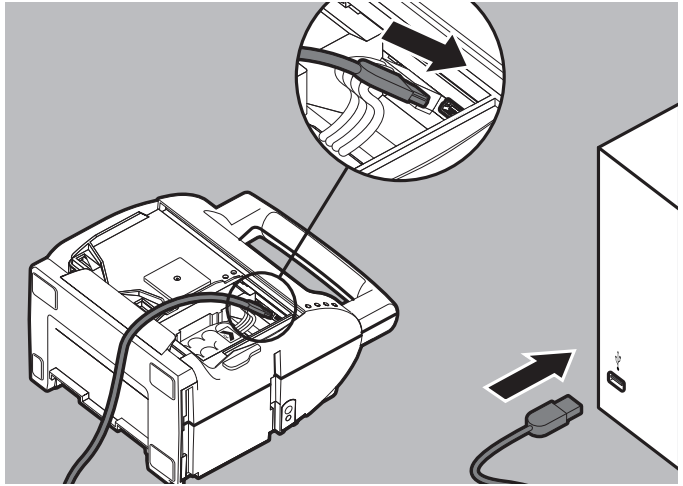3. Lift the battery compartment cover from underneath and remove it from the upper guide in a downward direction.
4. Put the device down on the lower part of the housing. The open battery compartment faces upward. The USB port is accessible.

5. Connect the USB cable to the USB port in the device.
6. Connect the USB cable to the PC.

*Result* The device is connected to the PC.

### 4.3 Start Device and Update Manager

- Requirements*
- Device and Update Manager is installed on the PC (see ["4.1 Install Device and Update Manager"](#), page 17).
  - The device is connected to the PC (see ["4.2 Connect the device to the PC"](#), page 17).
  - The device is switched off.
1. Double-click on file **Device and update manager.exe** to open it.

*Result* Device and Update Manager is started.

### 4.4 Read out device data

- Requirements* Device and Update Manager has been started (see ["4.3 Start Device and Update Manager"](#), page 19).

1. Select the **Device data & update** tab.
2. Select the **Read device data** button.  
The serial number and current firmware version of the device are read out.
3. If there is no connection to the device via Device and Update Manager: Briefly switch device on and then off again.

*Result* Device data are read out.

## 4.5 Perform firmware update

### *Requirements*

- Device data are read out (see "[4.4 Read out device data](#)", page 19).
  - A more recent firmware version is available in the login area of the WEINMANN Emergency home page.
1. Download the current firmware version from the login area of the WEINMANN Emergency home page.  
The software is available in the selected storage location in the form of a **hex** file.
  2. If the software is available in the form of a zip file: Unzip the software.

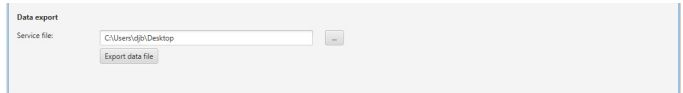
3. Select the **Device data & update** tab.
4. Select the storage location for the firmware update using the  button.
5. Select the **Execute firmware update** button.

*Result* The current firmware version is installed.

## 4.6 Read out service data

*Requirements* Device data are read out (see "4.4 Read out device data", page 19).

1. Select the **Device data & update** tab.

2. Select the storage location for the service file using the  button.
3. Select the **Export data file** button.  
A service file of device service data will be created and stored.
4. If necessary: Send the service file to WEINMANN Emergency or to specialist staff authorized by WEINMANN Emergency.


*Result* Service data are read out.

## 4.7 After use

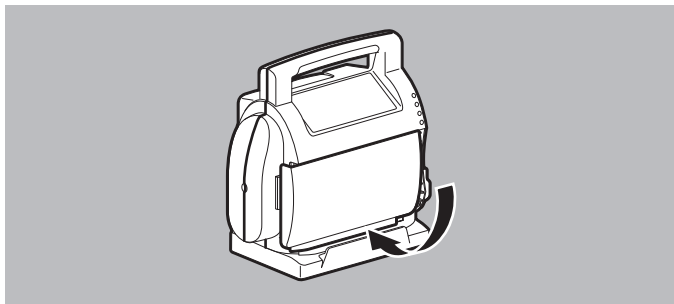
1. Remove the USB cable between the device and the PC.


2. Insert the battery compartment cover in the upper guide of the battery compartment.

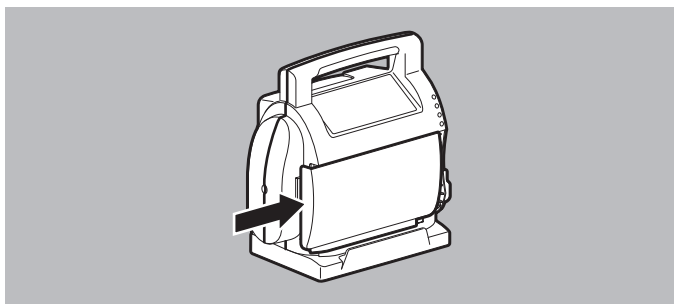3. Push the battery compartment cover into the lower guide of the battery compartment.

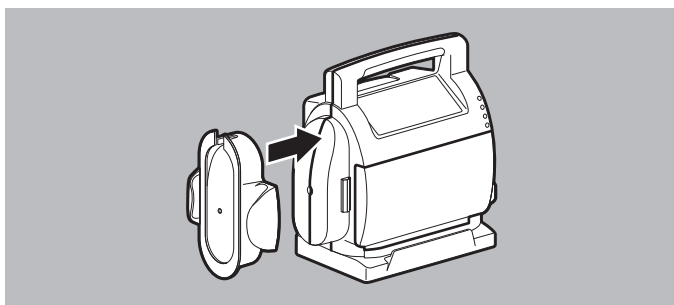4. Push the battery compartment cover to the right until you hear it engage.

5. Push the holder for suction tube with tube holder onto the device upward and sideways until you hear it engage.
6. Perform a function check in accordance with the Instructions for Use.

## 5 Faults

Fault	Cause	Remedy
No connection between device and Windows® PC	USB cable defective	Replace USB cable.
	USB interface on Windows® PC defective	Have PC repaired.
	USB interface on device defective	Have device repaired.
	No residual electrical capacity available in the device	Switch on the device for 5 s.
	Battery capacity $\leq 10\%$	Charge device.
Device and Update Manager cannot be opened	No current version of JAVA installed	Perform JAVA update.
	EXE file not unzipped from zip folder	Unzip file.
Unable to update firmware	Device data not read out	Read out device data.
	No firmware update stored in selected storage location	Select correct folder.
Unable to read out service data	Device data not read out	Read out device data.

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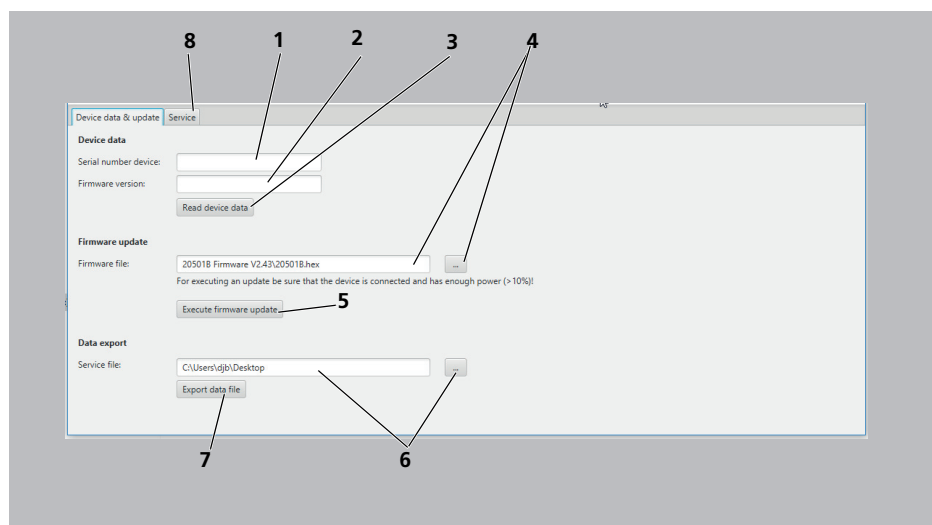
### 4.1 Installation du Device and Update Manager

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Medical Technology GmbH + Co. KG
ATMOS MedizinTechnik GmbH & Co. KG